

Клиническая документация по системе имплантатов ANKYLOS®

Система имплантатов ANKYLOS обладает рядом ключевых особенностей, в том числе фрикционным коническим соединением имплантата и абатмента, присущим системе горизонтальным смещением и микроструктурированной поверхностью плеча импланта. Данная система используется при различных клинических показаниях уже больше 25 лет.

Согласно опубликованным данным, установка имплантатов ANKYLOS — безопасная и предсказуемая процедура в обеих челюстях при таких показаниях, как реставрации одиночного зуба¹⁻¹², несъемные частичные или полные протезы¹¹⁻¹⁵ и съемные протезы^{11, 16-19}. Более того, имеются опубликованные клинические результаты для немедленной установки имплантатов в лунки удаленных зубов^{6, 8, 17, 20-22}, установки имплантатов в трансплантированную кость^{23, 24} и установки имплантатов с применением одноэтапной хирургической процедуры с немедленной нагрузкой^{5, 6-8, 13, 14, 16, 17, 20-22, 25-37}.

В клинических исследованиях со сроком последующего наблюдения от 1 до 8 лет продемонстрированы безопасность использования имплантатов ANKYLOS и высокие показатели приживаемости имплантатов — от 94 до 100 %^{3-11, 13, 14, 16-18, 20-36, 38-42}. Клиническую безопасность также подтверждает ретроспективное исследование более 12 500 имплантатов ANKYLOS со сроком документально подтвержденного клинического наблюдения до 20 лет⁴³.

В нескольких исследованиях^{1, 26-29, 44} документально подтверждена хорошая первичная стабильность имплантатов ANKYLOS. Среднее значение усилия фиксации составляет от 28,8 до 47,5 Нсм даже для имплантата диаметром Ø3,5 мм^{8, 26-28, 45}. Кроме того, отмечена высокая степень удовлетворенности пациентов^{2, 21}.

В опубликованных клинических исследованиях сообщается о среднем изменении уровня кости в области шейки имплантатов ANKYLOS после 1 года^{7, 9, 23, 46, 47} (от 0,01 до -1,32 мм), 2 лет^{13, 20, 26} (от +0,21 до -0,78 мм) и 3 лет¹² функционирования (-0,6 мм).

1. Morris HF, Ochi S, Orenstein IH, et al. AICRG, Part V: Factors influencing implant stability at placement and their influence on survival of ANKYLOS implants. *J Oral Implantol* 2004;30(3):162-70. [Abstract in PubMed](#)
2. Morris HF, Ochi S, Rodriguez A, et al. AICRG, Part IV: Patient satisfaction reported for ANKYLOS implant prostheses. *J Oral Implantol* 2004;30(3):152-61. [Abstract in PubMed](#)
3. Morris HF, Ochi S, Crum P, et al. AICRG, Part I: A 6-year multicentered, multidisciplinary clinical study of a new and innovative implant design. *J Oral Implantol* 2004;30(3):125-33. [Abstract in PubMed](#)
4. Doring K, Eisenmann E, Stiller M. Functional and esthetic considerations for single-tooth ANKYLOS implant-crowns: 8 years of clinical performance. *J Oral Implantol* 2004;30(3):198-209. [Abstract in PubMed](#)
5. Romanos GE, Nentwig GH. Single molar replacement with a progressive thread design implant system: a retrospective clinical report. *Int J Oral Maxillofac Implants* 2000;15(6):831-6. [Abstract in PubMed](#)
6. Degidi M, Daprile G, Nardi D, et al. Immediate provisionalization of implants placed in fresh extraction sockets using a definitive abutment: the chamber concept. *Int J Periodontics Restorative Dent* 2013;33(5):559-65. [Abstract in PubMed](#)
7. Abboud M, Koeck B, Stark H, et al. Immediate loading of single-tooth implants in the posterior region. *Int J Oral Maxillofac Implants* 2005;20(1):61-8. [Abstract in PubMed](#)
8. Degidi M, Nardi D, Daprile G, et al. Nonremoval of immediate abutments in cases involving subcrestally placed postextractive tapered single implants: A randomized controlled clinical study. *Clin Implant Dent Relat Res* 2013;E-pub March 6, doi:10.1111/cid.12051. [Abstract in PubMed](#)
9. Koutouzis T, Neiva R, Nonhoff J, et al. Placement of implants with platform-switched morse taper connections with the implant-abutment interface at different levels in relation to the alveolar crest: a short-term (1-year) randomized prospective controlled clinical trial. *Int J Oral Maxillofac Implants* 2013;28(6):1553-63. [Abstract in PubMed](#)
10. Rinke S, Roediger M, Eickholz P, et al. Technical and biological complications of single-molar implant restorations. *Clin Oral Implants Res* 2014;E-pub Mar 29, doi:10.1111/clr.12382. [Abstract in PubMed](#)
11. Nentwig GH. ANKYLOS implant system: concept and clinical application. *J Oral Implantol* 2004;30(3):171-7. [Abstract in PubMed](#)
12. Chou CT, Morris HF, Ochi S, et al. AICRG, Part II: Crestal bone loss associated with the ANKYLOS implant: loading to 36 months. *J Oral Implantol* 2004;30(3):134-43. [Abstract in PubMed](#)
13. Bressan E, Lops D. Conometric retention for complete fixed prosthesis supported by four implants: 2-years prospective study. *Clin Oral Implants Res* 2014;25(5):546-52. [Abstract in PubMed](#)
14. Romanos GE, Nentwig GH. Immediate versus delayed functional loading of implants in the posterior mandible: a 2-year prospective clinical study of 12 consecutive cases. *Int J Periodontics Restorative Dent* 2006;26(5):459-69. [Abstract in PubMed](#)
15. Rinke S, Ohi S, Ziebolz D, et al. Prevalence of periimplant disease in partially edentulous patients: a practice-based cross-sectional study. *Clin Oral Implants Res* 2011;22(8):826-33. [Abstract in PubMed](#)
16. Shaarawy MA, Abolross EM. The effect of varying implant position in immediately loaded implant-supported mandibular overdentures. *J Oral Implantol* 2013;39(3):345-54. [Abstract in PubMed](#)
17. Eccellente T, Piombino M, Piattelli A, et al. A new treatment concept for immediate loading of implants inserted in the edentulous mandible. *Quintessence Int* 2010;41(6):489-95. [Abstract in PubMed](#)
18. Frisch E, Ziebolz D, Ratka-Kruger P, et al. Double crown-retained maxillary overdentures: 5-year follow-up. *Clin Implant Dent Relat Res* 2013;E-pub May 18, doi:10.1111/cid.12087. [Abstract in PubMed](#)
19. Ahmad R, Abu-Hassan MI, Li Q, et al. Three dimensional quantification of mandibular bone remodeling using standard tessellation language registration based superimposition. *Clin Oral Implants Res* 2012;24(11):1273-9. [Abstract in PubMed](#)
20. Crespi R, Cappare P, Gherlone E. Radiographic evaluation of marginal bone levels around platform-switched and non-platform-switched implants used in an immediate loading protocol. *Int J Oral Maxillofac Implants* 2009;24(5):920-6. [Abstract in PubMed](#)
21. Eccellente T, Piombino M, Piattelli A, et al. Immediate loading of dental implants in the edentulous maxilla. *Quintessence Int* 2011;42(4):281-9. [Abstract in PubMed](#)
22. Romanos GE, May S, May D. Immediate loading of tooth-implant-supported telescopic mandibular prostheses. *Int J Oral Maxillofac Implants* 2012;27(6):1534-40. [Abstract in PubMed](#)
23. Koutouzis T, Fetner M, Fetner A, et al. Retrospective evaluation of crestal bone changes around implants with reduced abutment diameter placed non-submerged and at subcrestal positions: the effect of bone grafting at implant placement. *J Periodontol* 2011;82(2):234-42. [Abstract in PubMed](#)
24. Luo ZB, Zhang QB, Zhang ZQ, et al. Performance of coralline hydroxyapatite in sinus floor augmentation: a retrospective study. *Clin Oral Investig* 2013;17(9):2003-10. [Abstract in PubMed](#)
25. Abboud M, Wahl G, Guirado JL, et al. Application and success of two stereolithographic surgical guide systems for implant placement with immediate loading. *Int J Oral Maxillofac Implants* 2012;27(3):634-43. [Abstract in PubMed](#)
26. Degidi M, Nardi D, Piattelli A. Prospective study with a 2-year follow-up on immediate implant loading in the edentulous mandible with a definitive restoration using intra-oral welding. *Clin Oral Implants Res* 2010;21(4):379-85. [Abstract in PubMed](#)
27. Degidi M, Nardi D, Piattelli A. One abutment at one time: non-removal of an immediate abutment and its effect on bone healing around subcrestal tapered implants. *Clin Oral Implants Res* 2011;22(11):1303-07. [Abstract in PubMed](#)
28. Degidi M, Nardi D, Sighinolfi G, et al. Immediate rehabilitation of the edentulous mandible using ANKYLOS SynCone telescopic copings and intraoral welding: A pilot study. *Int J Periodontics Restorative Dent* 2012;32(6):e189-94. [Abstract in PubMed](#)
29. Romanos GE, Gaertner K, Nentwig GH. Long-term evaluation of immediately loaded implants in the edentulous mandible using fixed bridges and platform shifting. *Clin Implant Dent Relat Res* 2013;E-pub Jan 12, doi:10.1111/cid.12032. [Abstract in PubMed](#)
30. Donovan R, Fetner A, Koutouzis T, et al. Crestal bone changes around implants with reduced abutment diameter placed non-submerged and at subcrestal positions: a 1-year radiographic evaluation. *J Periodontol* 2010;81(3):428-34. [Abstract in PubMed](#)
31. May D, Romanos GE. Immediate implant-supported mandibular overdentures retained by conical crowns: A new treatment concept. *Quintessence Int* 2002;33(1):5-12. [Abstract in PubMed](#)
32. Romanos GE, May S, May D. Treatment concept of the edentulous mandible with prefabricated telescopic abutments and immediate functional loading. *Int J Oral Maxillofac Implants* 2011;26(3):593-7. [Abstract in PubMed](#)
33. Romanos GE, May S, May D. Implant-supporting telescopic maxillary prostheses and immediate loading. *Clin Implant Dent Relat Res* 2014;16(3):412-8. [Abstract in PubMed](#)
34. Romanos GE, Nentwig GH. Immediate loading using cross-arch fixed restorations in heavy smokers: nine consecutive case reports for edentulous arches. *Int J Oral Maxillofac Implants* 2008;23(3):513-9. [Abstract in PubMed](#)
35. Romanos GE, Gaertner K, Aydin E, et al. Long-term results after immediate loading of platform-switched implants in smokers versus nonsmokers with full-arch restorations. *Int J Oral Maxillofac Implants* 2013;28(3):841-5. [Abstract in PubMed](#)
36. Romanos GE, Malmstrom H, Feng C, et al. Immediately loaded platform-switched implants in the anterior mandible with fixed prostheses: A randomized, split-mouth, masked prospective trial. *Clin Implant Dent Relat Res* 2013;E-pub April 5, doi:10.1111/cid.12065. [Abstract in PubMed](#)
37. Wittwer G, Adeyemo WL, Wagner A, et al. Computer-guided flapless placement and immediate loading of four conical screw-type implants in the edentulous mandible. *Clin Oral Implants Res* 2007;18(4):534-9. [Abstract in PubMed](#)
38. Morris HF, Winkler S, Ochi S, et al. A new implant designed to maximize contact with trabecular bone: survival to 18 months. *J Oral Implantol* 2001;27(4):164-73. [Abstract in PubMed](#)
39. Romanos GE, Nentwig GH. Immediate functional loading in the maxilla using implants with platform switching: five-year results. *Int J Oral Maxillofac Implants* 2009;24(6):1106-12. [Abstract in PubMed](#)
40. Morris HF, Ochi S, Plezia R, et al. AICRG, Part III: The influence of antibiotic use on the survival of a new implant design. *J Oral Implantol* 2004;30(3):144-51. [Abstract in PubMed](#)
41. Romanos GE, Aydin E, Gaertner K, et al. Long-term results after subcrestal or crestal placement of delayed loaded implants. *Clin Implant Dent Relat Res* 2013;E-pub May 17, doi:10.1111/cid.12084. [Abstract in PubMed](#)
42. Frisch E, Ratka-Kruger P, Wenz HJ. Unsplinted implants and teeth supporting maxillary removable partial dentures retained by telescopic crowns: a retrospective study with >6 years of follow-up. *Clin Oral Implants Res* 2014;E-pub April 17, doi:10.1111/clr.12407. [Abstract in PubMed](#)
43. Krebs M, Schmeiger K, Neumann K, et al. Long-term evaluation of ANKYLOS dental implants, Part I: 20-year life table analysis of a longitudinal study of more than 12,500 implants. *Clin Implant Dent Relat Res* 2013;E-pub Sep 2013, doi:10.1111/cid.12154. [Abstract in PubMed](#)
44. Morris HF, Winkler S, Ochi S. The ANKYLOS endosseous dental implant: assessment of stability up to 18 months with the Periotest. *J Oral Implantol* 2000;26(4):291-9. [Abstract in PubMed](#)
45. Rabel A, Kohler SG, Schmidt-Westhausen AM. Clinical study on the primary stability of two dental implant systems with resonance frequency analysis. *Clin Oral Investig* 2007;11(3):257-65. [Abstract in PubMed](#)
46. Degidi M, Daprile G, Nardi D, et al. Buccal bone plate in immediately placed and restored implant with Bio-Oss(R) collagen graft: a 1-year follow-up study. *Clin Oral Implants Res* 2013;24(11):1201-5. [Abstract in PubMed](#)
47. Shin YK, Han CH, Heo SJ, et al. Radiographic evaluation of marginal bone level around implants with different neck designs after 1 year. *Int J Oral Maxillofac Implants* 2006;21(5):789-94. [Abstract in PubMed](#)