

Lucitone Digital Print Denture

Troubleshooting Guide for Clinicians & Laboratories

Denture Try-In

Posterior Teeth	Likely Cause	Next Steps
Occlusion at rest is misaligned	<ul style="list-style-type: none"> Centric not captured correctly Design not articulated Bite registration material was not used 	<ul style="list-style-type: none"> Modify Try-In using new occlusal records ensuring centric registration is repeatable Fabricate a bite registration Reprint Try-In
Vertical is open	<ul style="list-style-type: none"> Bite registration inaccurate Centric not captured correctly 	<ul style="list-style-type: none"> Adjust any areas that may be impeding proper closure of bite Add wax to close and re-establish correct vertical Rescan, redesign, and reprint
Vertical is closed	<ul style="list-style-type: none"> Bite registration is not accurate Burn-through on the custom tray Centric not captured correctly Insufficient vertical space 	<ul style="list-style-type: none"> Adjust the Try-In for intaglio high spots Establish centric relationship New impression, take registration bite Rescan, redesign, and reprint
Wrong occlusal strategy	<ul style="list-style-type: none"> Prescription error Design error 	<ul style="list-style-type: none"> Repeat Try-In
Anterior Teeth	Likely Cause	Next Steps
Midline is off	<ul style="list-style-type: none"> Prescription error Design error 	<ul style="list-style-type: none"> Adjust and evaluate if Try-In can proceed to final
Overbite / Overjet	<ul style="list-style-type: none"> Design error Case class problem Clinician did not recommend classification set-up 	<ul style="list-style-type: none"> Verify class set-up with clinician Redesign and evaluate if case can proceed to final denture Repeat Try-In if overbite/overjet is severely off
Length of central short / long	<ul style="list-style-type: none"> Incorrect prescription Omitted or inaccurate incoming records Smile line information missing Design error 	<ul style="list-style-type: none"> Review in-coming records for accuracy Adjust design and evaluate if case can proceed to final
Canine positioning off	<ul style="list-style-type: none"> Incorrect prescription Omitted or inaccurate incoming records Distal of canines not marked on wax rim Design error 	<ul style="list-style-type: none"> Review in-coming records for accuracy Adjust design and evaluate if case can proceed to final
Phonetics issue	<ul style="list-style-type: none"> Inclination of the incisal teeth 	<ul style="list-style-type: none"> Clinician to confirm phonetic issue: <ol style="list-style-type: none"> Difficulty with S & H sounds - adjust teeth either facially or lingually Difficulty with F & V sounds - adjust teeth higher or lower Make adjustments, reprint Try-In
Esthetics & Other	Likely Cause	Next Steps
Lip support inadequate / too full	<ul style="list-style-type: none"> Check bite rim, review patient assessment from profile view 	<ul style="list-style-type: none"> Clinician to provide adjustment (wax) Modify teeth position in design (if required), redesign, and evaluate if case can proceed to final
Fit / Retention / Poor suction	<ul style="list-style-type: none"> Review impression accuracy Confirm if design included bar supports 	<ul style="list-style-type: none"> Repeat impressions, rescan, new models, redesign with bars, reprint Try-In
Discomfort / High spots	<ul style="list-style-type: none"> Final impression not adjusted 	<ul style="list-style-type: none"> Add material in the design Adjust Try-In, reprint, evaluate if case can move to final

Lucitone Digital Print Denture

Troubleshooting Guide for Clinicians & Laboratories

Final Denture

Posterior Teeth	Likely Cause	Next Steps
Occlusion at rest is misaligned	<ul style="list-style-type: none"> • Centric not captured correctly • Design not articulated • Bite registration material was not used 	<ul style="list-style-type: none"> • Clinician modify final. If modifications are acceptable for the patient, rescan the denture and upload the files to capture the new design records. • If modifications are not acceptable for the patient, take new occlusal records (if required). Lab to rescan, update design, and reprint final.
Vertical is open	<ul style="list-style-type: none"> • Posterior teeth are not properly equilibrated • Check final workmanship of tooth bonding procedure • Recheck previous scans and design 	<ul style="list-style-type: none"> • Adjust any areas impeding proper closure of the bite • Add wax to close and re-establish the correct vertical • Rescan, redesign, reprint
Vertical is closed	<ul style="list-style-type: none"> • Design error • Check vertical space 	<ul style="list-style-type: none"> • Clinician to adjust high spots of the intaglio surface • If acceptable for the patient, rescan the denture and upload the files to capture the new design records • If not acceptable, order a redesign and new Try-In
Wrong occlusal strategy	<ul style="list-style-type: none"> • Prescription error • Design error 	<ul style="list-style-type: none"> • Repeat Try-In
Anterior Teeth	Likely Cause	Next Steps
Midline is off	<ul style="list-style-type: none"> • Prescription error • Design error 	<ul style="list-style-type: none"> • Adjust and evaluate if design is printable or if a new Try-In is needed
Overbite / Overjet	<ul style="list-style-type: none"> • Design error • Case class problem 	<ul style="list-style-type: none"> • Review Try-In and previous scan files for accuracy • Redesign, reprint final
Length of central short / long	<ul style="list-style-type: none"> • Design error • Workmanship error 	<ul style="list-style-type: none"> • Review original scans and designs for errors • Confirm records with Clinician • Make adjustments to designs and reprint
Canine positioning off	<ul style="list-style-type: none"> • Design error • Workmanship error 	<ul style="list-style-type: none"> • Review original scans and designs for errors • Clinician to confirm records • Make adjustments to designs and reprint
Phonetics issue	<ul style="list-style-type: none"> • Inclination of the incisal teeth 	<ul style="list-style-type: none"> • Clinician to confirm phonetic issue: <ol style="list-style-type: none"> 1) Difficulty with S & H sounds - adjust teeth either facially or lingually 2) Difficulty with F & V sounds - adjust teeth higher or lower • Make adjustments, reprint denture

Dentsply Sirona | 800-243-1942 | dentsplysirona.com

DP-0000461 Rev. 2 (07/2021)